

Easy Series

Easy Extraction Basic Set



Direction for use

Periotome



Forcep



Luxator



- “Easy Series” consist of independent product such as “Easy Periotome”, “Easy Forcep” and “Easy Luxator” that you may purchase separately. You may also purchase them in a set product.





01 Easy Periotome


This product is used for minimizing damage to the alveolar bone as it helps cut the periodontal ligament to detach teeth from the periodontal ligament for implant immediately after tooth extraction or general tooth extraction.

Cautions

- Easy Periotome is used for cutting off only the periodontal ligament.
Do not lift it up or twist with excessive force such as Elevator.

Easy Periotome

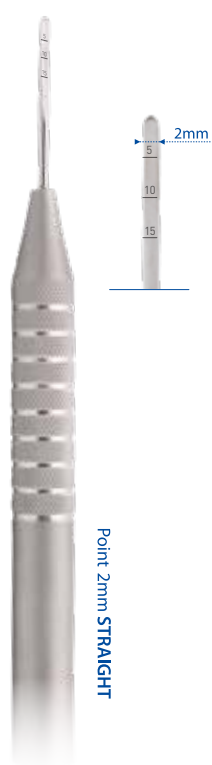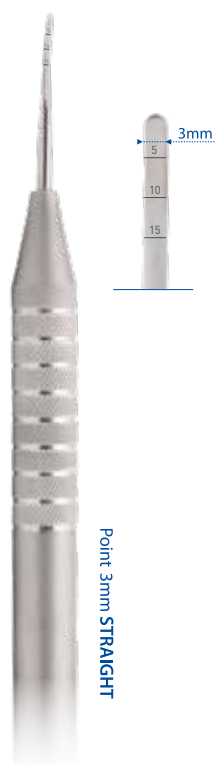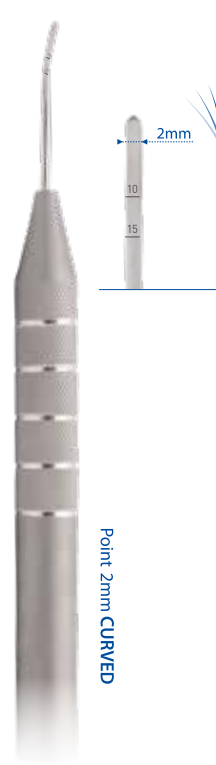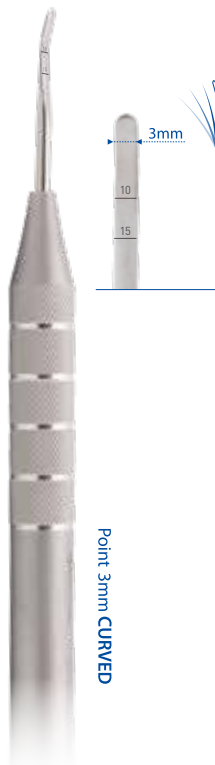 <p>Point 2mm STRAIGHT</p>	 <p>Point 3mm STRAIGHT</p>	 <p>Point 2mm CURVED</p>	 <p>Point 3mm CURVED</p>
Easy Periotome 2mm [STRAIGHT] CODE NO. 3305	Easy Periotome 3mm [STRAIGHT] CODE NO. 3306 ● Included in the Easy Extraction Basic Set	Easy Periotome 2mm [CURVED] CODE NO. 3307	Easy Periotome 3mm [CURVED] CODE NO. 3308 ● Included in the Easy Extraction Basic Set

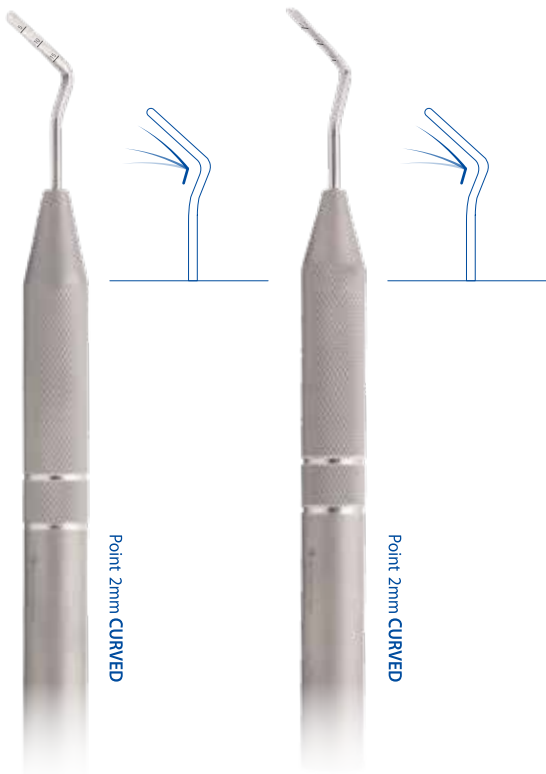

Easy Periotome

2mm [CURVED]
CODE NO. **3311**

●
**Exclusive use for
Posterior Part**

Easy Periotome

2mm [CURVED]
CODE NO. **3312**

●
**Exclusive use for
Posterior Part**



Flexible Type

- Easy Periotome adopts the flexible type which can be bent freely, and ensures approach to the tooth root, so that you can cut off the periodontal ligament very easily.



Easy Periotome Set



02 Easy Forceps

Easy Forcep provides a new specialized tip design, not the existing way to hold a tooth for extraction, but a new approach to penetrate into the inside of periodontal tissue for tooth extraction. It lessens the luxation effect and possibility of root residues while minimizing damage to the alveolar bone.




Notice

- Our Specialized design enables easier and faster tooth extraction.

Included in the
Easy Extraction Basic Set




**Maxillary
Anterior Tooth**
CODE NO. 3251

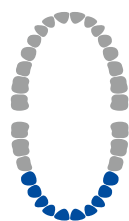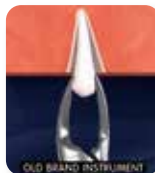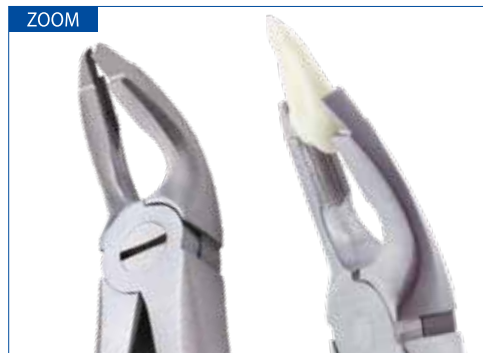

**Mandibular
Anterior Tooth**
CODE NO. 3252



Included in the
Easy Extraction Basic Set

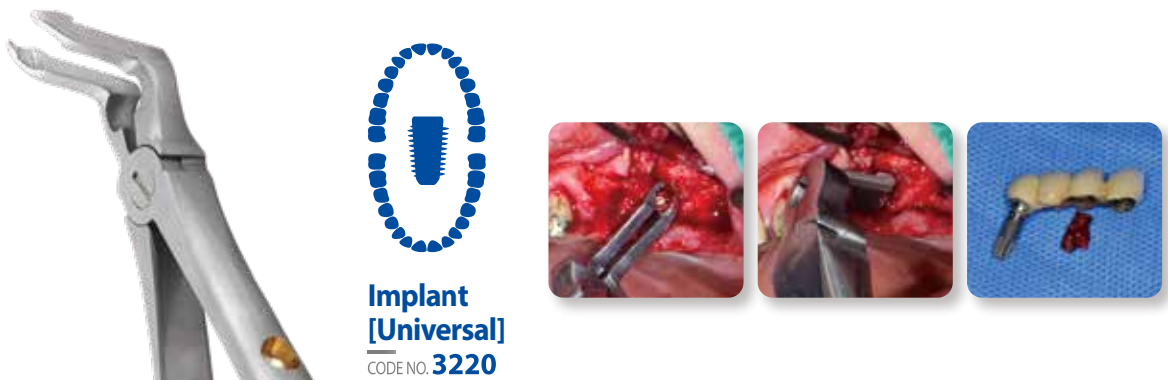

03 Easy Luxator

It is a New Easy Luxator which has flexible and thin blade unlike other Luxator. Due to this structural feature, the tip of the luxator easily digs into the PDL space. So the distortion of teeth from the alveolar bone can be induced with even a small force. It also enables Atraumatic extraction because it minimizes the damage of surrounding soft tissue and alveolar bone.



Easy Luxator

3mm [STRAIGHT]

CODE NO. **3111 & 3112**




Easy Luxator

4mm [STRAIGHT]

CODE NO. **3113 & 3114**

●
Included in the
Easy Extraction Basic Set

It is made of a special alloy, so it has elasticity and is thinner than other Luxator.
It has the advantage of free from breakage, preserving the shape of the skeleton, and minimizing tissue damage.

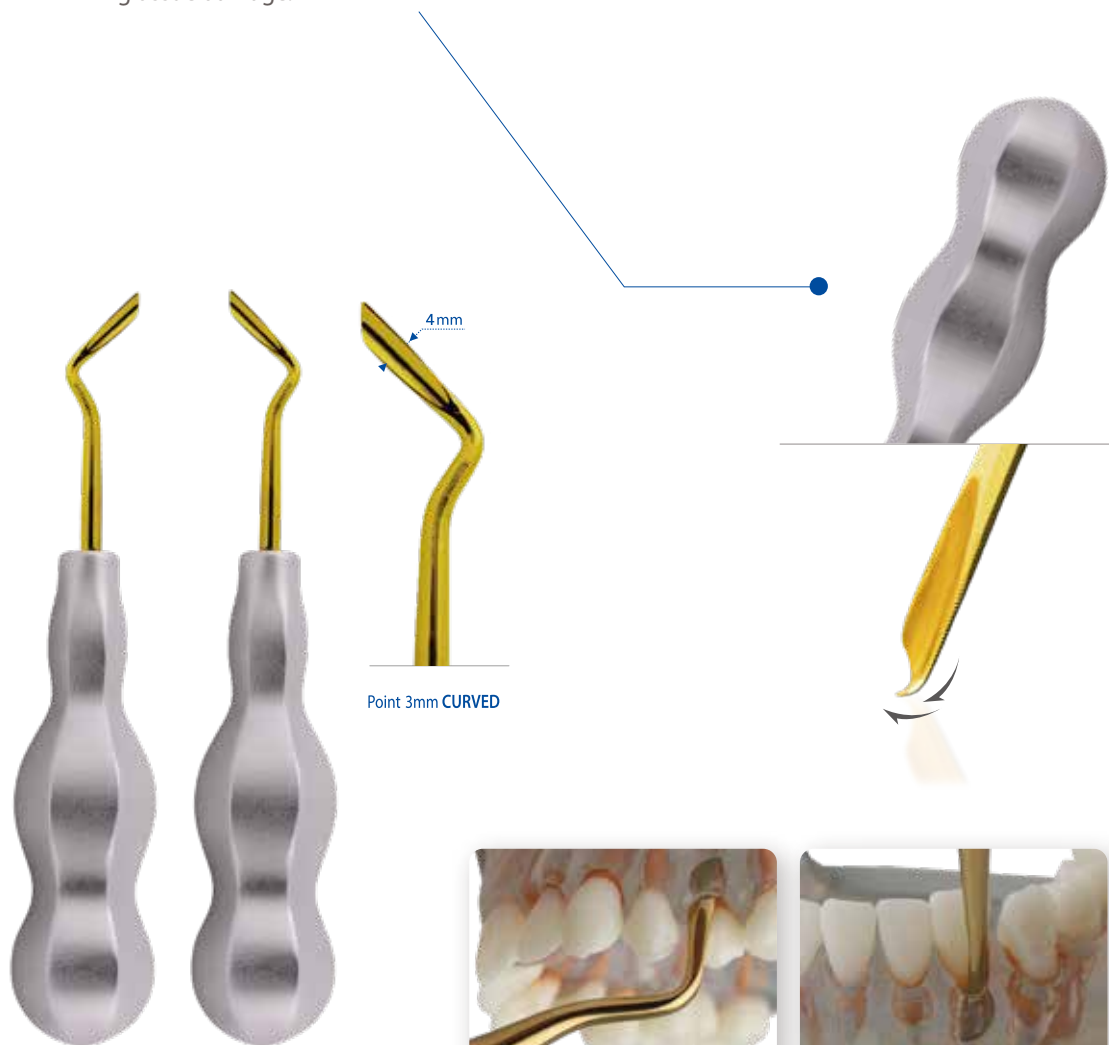

Easy Luxator

3mm [CURVED]

CODE NO. **3115 & 3116**

※ Caution: Do not lift it up or twist with excessive force such as Elevator

04 Easy Sinus Curette & Surgical Curette

Flexible Sinus Lift instruments increase the sensitiveness during operation and reduce the risk of perforation in the sinus membrane.

EASY SINUS CURETTE is made from more flexible and special materials than those of the existing products to provide a role of buffering in preventing the membrane from troubles while the sinus membrane is being serrated.

Notice

- Stable operation is helped with the design to feel the sensitiveness of the operator.
- Easy approach to the maxillary sinus is ensured with the specialized SINUS CURETTE design.
- The risk of perforation is substantially reduced.



CODE NO. **7096F** (Set)



SINUS CURETTE 1
CODE NO. **7091F**



SINUS CURETTE 2
CODE NO. **7092F**



**SINUS
CURETTE 3**
CODE NO. **7093F**



**SINUS
CURETTE 4**
CODE NO. **7094F**

As the tip part is made by tiny saw,
it is easy to remove the stick
foreign material by scraping
quickly and easily.



**Surgical
Curette [Saw]**
LUCAS86-1 / 3,5mm
CODE NO. **4004**

Included in the
Easy Extraction Basic Set

Not included in the
Easy Sinus Curette Set