



ZENEX Implant System

CONTENTS

01

Zenex Mini Kit

Surgical Tray & Instrument006

Point Drill007

Twist Drill007

Drill Stopper008

Drill Extension008

Torque Ratchet008

Handpiece Connector009

Ratchet Adapter010

Hex Driver 1.25011


Connector Pin011

Zenex Mini Kit Drilling Sequence012


04

ZENEX Mini Kit

ZENEX Mini, Surgical Tray and Instrument

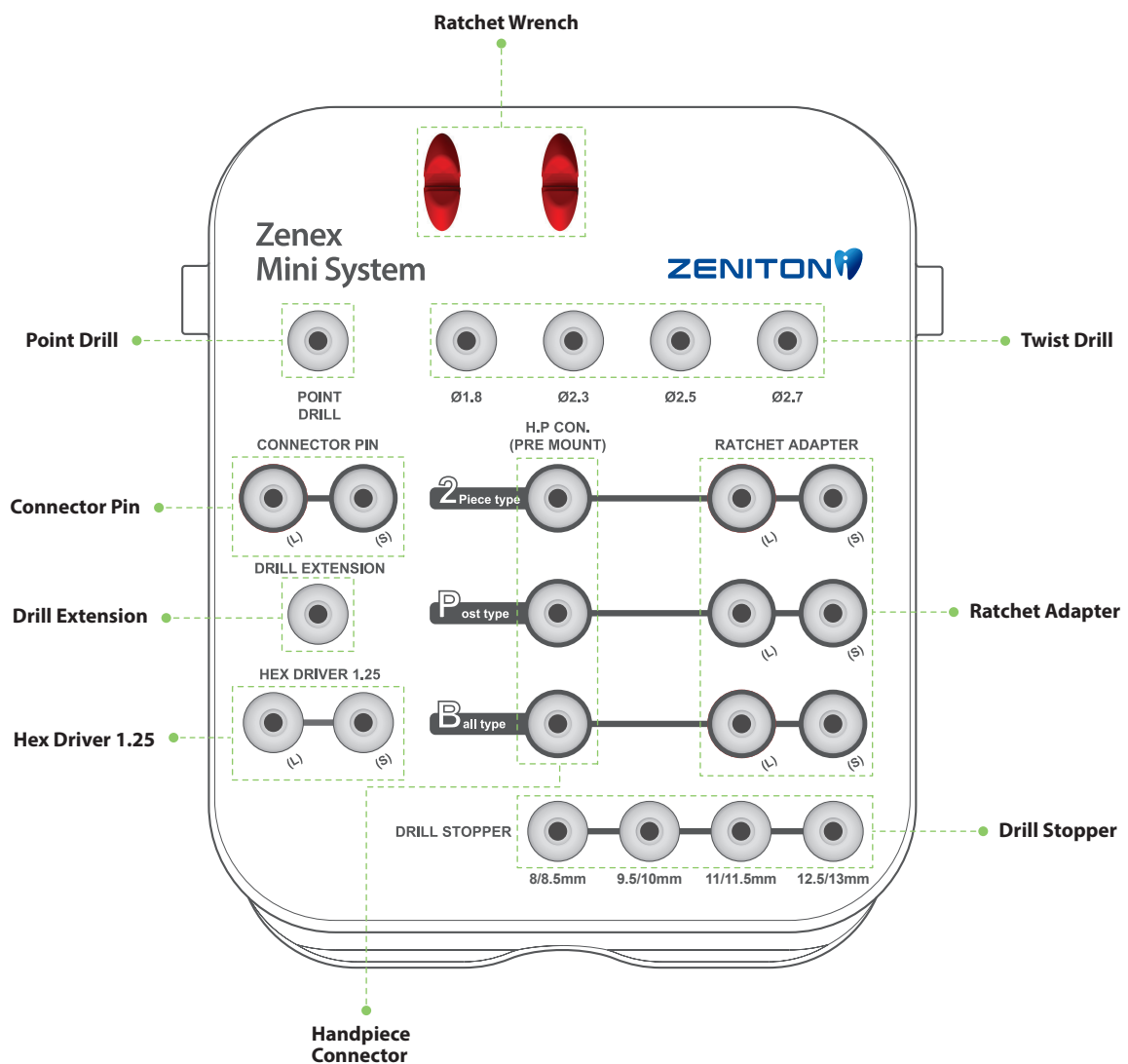

Point Drill

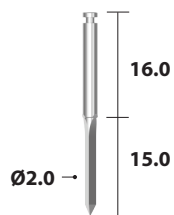

D	Ø2.0
-	ZTPTD20

- The density of bone can be notified during initial drilling
- Easily mark and form a hole on the bone for initial drilling
- Laser marking to show the hole depth

Twist Drill



Ø1.8



Ø2.0



Ø2.3



Ø2.5



Ø2.7

D	Ø1.8	Ø2.0	Ø2.3	Ø2.5	Ø2.7
-	ZTMD18S	ZTMD20S	ZTMD23S	ZTMD25S	ZTMD27S

- Laser marking to show hole depth
- Twisted structure for excellent cutting ability
- TiN coating for anti-abrasion and anti-corrosion effect


Drill Stopper



Applied Height For Fixture	8.0mm	9.5mm	11.0mm	12.5mm
Height of Stopper	8.5mm	10.0mm	11.5mm	13.0mm
-	ZTMDS08	ZTMDS10	ZTMDS11	ZTMDS13

- Select according to the length
- Recognize by color
- Could connect to the drill

Drill Extension

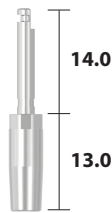

ZTDESP

- Extension of drills for hand pieces
- Connect the flat surface of the drill with the flat surface of the drill extension
- No excessive power is allowed for the connection


Torque Ratchet



ZATW40S

- Use when control the insertion location of implant or tight the abutment, screw etc
- Could give 10, 20, 30 Ncm or limitless torque
- The last line is usually means about 40Ncm
- Pull the bar and locate the the center of bar at the wanted torque value
- Washing and make sterilization after use it

Handpiece Connector

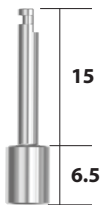

[Zenex Mini Submerged]

ZTHC350



[Zenex Mini Post]

ZTMHCP30




[Zenex Mini Ball]

ZTMHCB24


- Used for hand piece in the case of installation of fixture with the mount of the connector pin
- Internal holing function

Ratchet Adapter

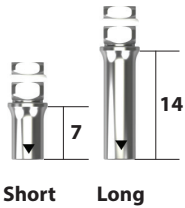


[Zenex Mini Submerged]

Type	Short	Long
-	ZTRA350SS	ZTRA350LS

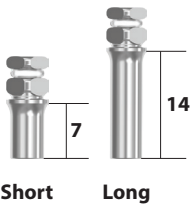

[Zenex Mini Post]

Type	Short	Long
-	ZTMPRA30S	ZTMPRA30L

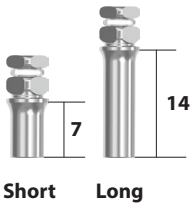


[Zenex Mini Ball]

Type	Short	Long
-	ZTMBRA24S	ZTMBRA24L

- Used for the ratchet in the case of installation of fixture with the mount or connector pin
- Internal holing function

Hex Driver 1.25

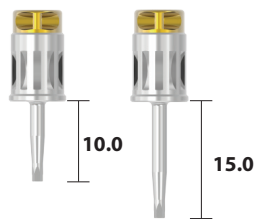

[Zenex Mini Submerged]

Type	Short	Long
-	ZTHD125S	ZTHD125L

- Hex driver operated by hand
- Hex 1.25

Connector Pin






[Zenex Mini Submerged]

Type	Short	Long
-	ZTCP17S	ZTCP17L

- Grooves to show the height

ZENEX Mini Kit, Drilling Sequence.

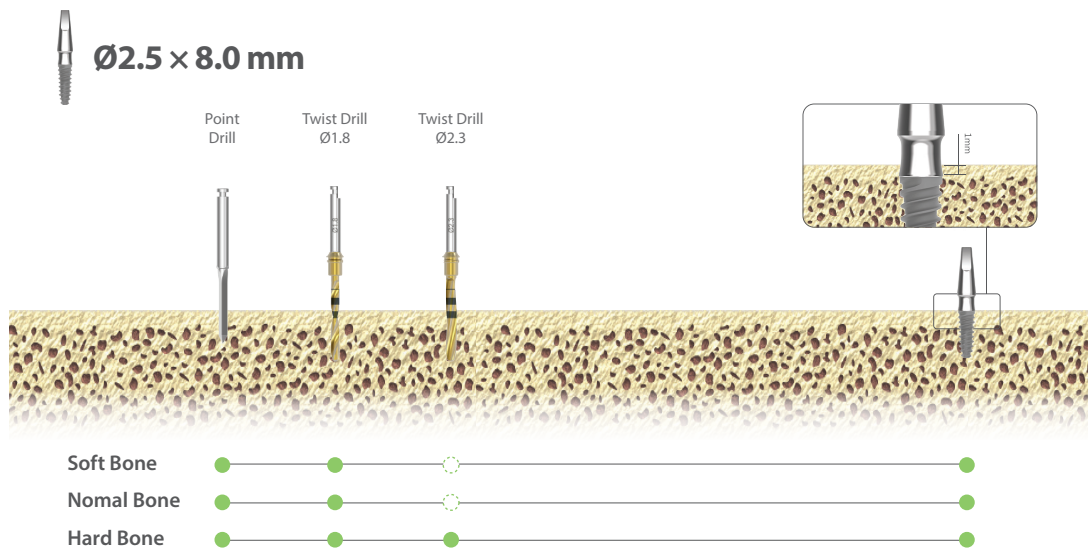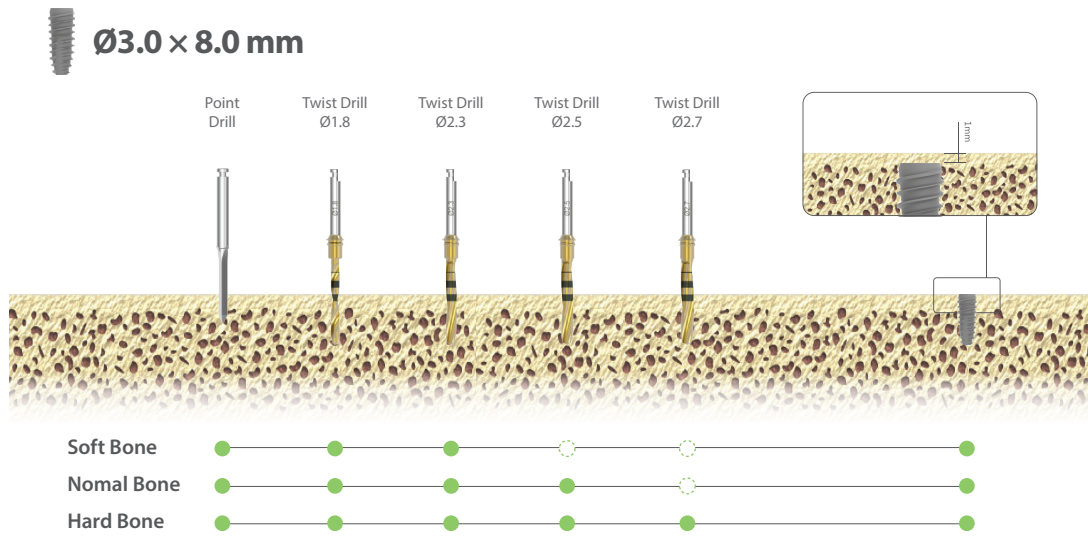

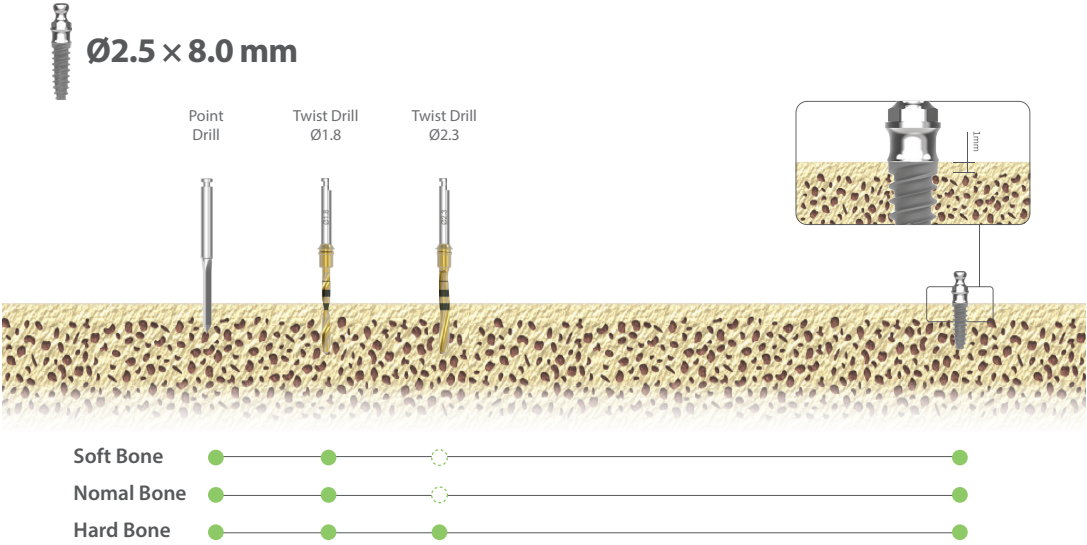

MEMO



HQ & Factory
1, 2Dong, 26-32, Suworam 4-gil, Seotan-myeon, Pyeongtaek-si,
Gyeonggi-do, Republic of Korea

Sales Office
#1013, #1014 Central-Tower II, 158, Magokseo-ro, Gangseo-gu, Seoul,
Republic of Korea
Tel. +82-2-3664-3666
Fax. +82-31-662-0658